

ZOLL AED 3™ BLS



ZOLL®



*High-quality CPR
for all your patients*

HIGH QUALITY WHEN AND WHERE YOU NEED IT MOST

High-quality CPR is the cornerstone of a system of care that can optimize resuscitation outcomes. The American Heart Association (AHA) and European Resuscitation Council (ERC) 2015 Guidelines underscore just how important CPR quality is for a successful resuscitation program. To deliver high-quality CPR, providers need to focus on depth, rate, limited interruptions, and release. ZOLL offers the same unique CPR feedback technology in our AEDs as we do with our manual defibrillators, allowing seamless care for patients, no matter where they may encounter cardiac issues in the hospital.

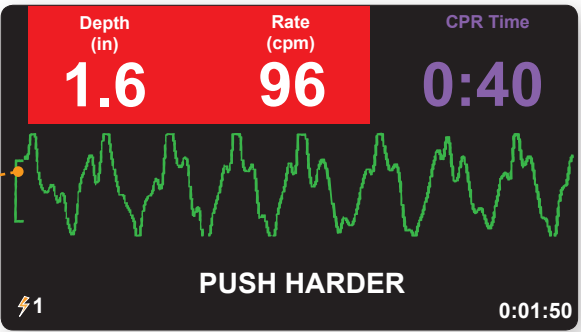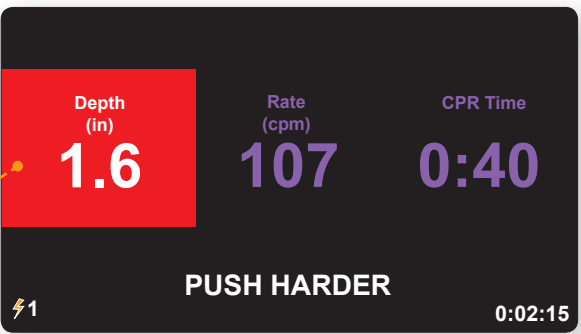

More than CPR. This is Real CPR Help.

The ZOLL AED 3 BLS features Real CPR Help®, which coaches on depth and rate of compressions for high-quality CPR. Not only can clinicians who routinely struggle with achieving proper depth and/or rate benefit from CPR coaching, but laypeople, who have very limited training, could benefit immensely. For the patient, it provides a greater chance of surviving sudden cardiac arrest (SCA).

Synergy with manual ZOLL defibrillators

Chances are high that a clinically trained staff member will be either the first or second responder in a hospital setting. If they are already familiar with the CPR Dashboard™ on R Series® or X Series®, then the dashboard of the ZOLL AED 3 BLS will be intuitive to use and valuable in helping them achieve high-quality CPR. This familiarity can bring speed of therapy to the patient.


Faster time-to-charge means faster defibrillation, a key metric in the AHA's Chain of Survival. With one of the fastest preshock pauses on the market at 8 seconds with a fresh battery, the ZOLL AED 3 assures clinicians they are delivering the fastest defibrillation shock available.



One pad for all patients, including pediatrics

The CPR Uni-padz™ are truly universal. No need to keep an extra set of pediatric pads on hand in the event the patient is a child. This speeds the therapy to the patient. No time-consuming decision making. If the patient is a child, just press the Child Selection button.

When Child Mode is selected, the metronome will sound when compressions are detected.

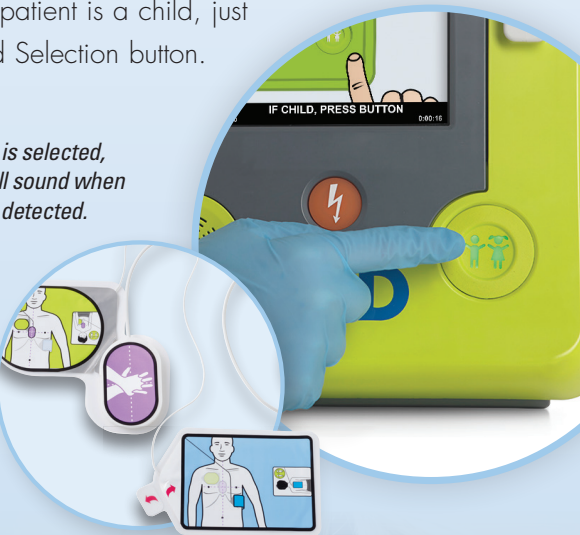

Ready for Tomorrow

You can upload clinical event files over WiFi to be viewed in ZOLL's RescueNet® Code Review software program. Files can also be sent automatically to ZOLL's newer software program, RescueNet CaseReview™, for viewing, parsing, and pulling metrics, including a CPR Summary Report. CaseReview will also store clinical event files from other ZOLL devices, such as R Series and X Series.

Network clock synchronization for the beginning and end of daylight saving time occurs whenever the ZOLL AED 3 communicates with the hospital WiFi system, ensuring your AED is always using the correct timestamp should an event occur and the AED is needed for a resuscitation.



The CPR quality can be uploaded after the event via WiFi and reviewed quickly with the code team to evaluate and identify areas of improvement.



ZOLL AED 3

- CAFETERIA
- PARKING GARAGE
- EXECUTIVE OFFICES

EXAMPLES OF LOCATIONS INCLUDE:

ZOLL AED 3 BLS

- DIALYSIS
- PHLEBOTOMY
- STRESS TEST LAB

Other Members of the ZOLL AED 3 Family

The ZOLL AED 3 is nearly identical to the ZOLL AED 3 BLS. It uses the same batteries and electrodes, and is WiFi enabled. It is designed with the lay rescuer in mind. It's a great choice for the hospital parking garage or general areas, where the first responder is not likely to be from the hospital's clinical staff.

The ZOLL AED 3 is intuitive to use and walks the responder through the rescue, from pad placement to calling for help. Of course, it also provides real-time feedback on chest compressions thanks to Real CPR Help.

Whether your hospital wants to place AEDs in locations where an experienced clinician is nearby or in public areas where someone from the general public might be responding, ZOLL offers you two ZOLL AED 3 choices—one suited for BLS trained personnel, the other for lay rescuers. No matter which you choose, you can be sure that the same quality and commitment you have come to expect from ZOLL in your clinical areas will be consistent in non-clinical ones as well.

ZOLL MEDICAL CORPORATION

An Asahi Kasei Group Company | 269 Mill Road | Chelmsford, MA 01824 | 978-421-9655 | 800-804-4356 | www.zoll.com

Copyright © 2017 ZOLL Medical Corporation. All rights reserved. CaseReview, Real CPR Help, RescueNet, Uni-padz, R Series, X Series, ZOLL, and ZOLL AED 3 are trademarks or registered trademarks of ZOLL Medical Corporation in the United States and/or other countries. All other trademarks are the property of their respective owners.

ZOLL AED 3 is not available for sale in the United States. The product has not received regulatory clearance/approval by the Food and Drug Administration.

Printed in U.S.A.
MCN HP 1704 0250

For subsidiary addresses and fax numbers, as well as other global locations, please go to www.zoll.com/contacts.

ZOLL®