


New
NeoPockets[™]
 Personal Incubator Organizer

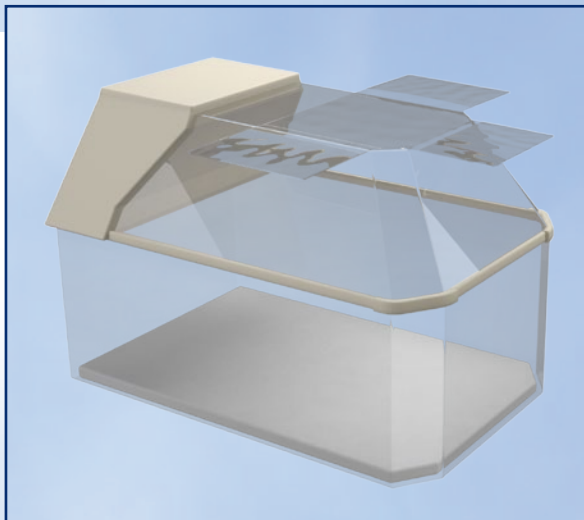

FEATURES AND BENEFITS

- Apply and remove repeatedly with no adhesive residue
- Crystal clear vinyl allows for visibility of patient
- Adheres to most smooth surfaces
- Three pockets for:
 - Patient info
 - Thermometer
 - Family photos and more
- Contours to a variety of incubator models
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

Cat No.	Item	Qty/Unit
N7325	NeoPockets	20/box



Made in USA



Clinician Invented

Patent Pending

NeoPockets™

Personal Incubator Organizer

INTENDED USE

Intended to store miscellaneous information and accessories.

DIRECTIONS FOR USE

Step 1

Unfold NeoPockets.

Step 2

Peel away from paper backing; discard backing.

Step 3

Lay middle section on top of incubator and align edges.

Step 4

Fold sides and front flap down over incubator

Step 5

Remove or reposition as needed.

GENERAL INFORMATION

Product Materials

Static cling vinyl

Made with natural rubber latex: No

Made with plasticizer DEHP: No

Size

25 in. x 16 in. (63.5 cm x 40.5 cm)

Pocket 1: 6.5 in. x 5 in. (165 mm x 127 mm)

Pocket 2: 2.3 in. x 5 in. (58 mm x 127 mm)

Pocket 3: 7.8 in. x 5.5 in. (198 mm x 140 mm)

Pocket 4 8.25 in. x 6.5 in. (209.5 x 165 mm)

CAUTIONS & WARNINGS

Discard if static cling becomes ineffective.

Distribute weight evenly.



For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com

Neotech Products, Inc. • 28430 Witherspoon Pkwy, Valencia, CA 91355